

# Clipper Vac® Attachment

## Compatible with Aesculap® FAV5



**Step 1** Turn your clipper upside-down and remove the four (4) visible screws.

**Note:** *Keep track of where the long screws and short screws are coming from as you will need to replace these in the same place once the new upper housing is attached.*

**Step 2** Remove the existing upper housing of the clipper.

**Step 3** Place the new housing with the Clipper Vac® Attachment in its place and replace the four (4) screws you removed previously.

**Note:** *DO NOT OVER-TIGHTEN THE SCREWS! Doing so risks stripping the inside of the clipper.*

M.D.C. Romani, Inc./Clipper Vac® is not affiliated with, authorized by or endorsed in any way by Aesculap®.